

## **Homefill Instructions**

### **Before Putting Tank on Homefill**

- The green regulators/conserving devices never need to come off.
  - All oxygen tanks should have their own regulator. Please contact us if not.
  - These are to be positioned where the 2 prongs inside the regulator are placed inside the 2 holes on the tank and screwed into the tank divot holding place.
- The tank needs to be less than halfway full.
  - If the tank is over halfway full, the homefill will not try to refill it.
- The tank needs to be fully shut.
  - The silver handle at top needs to be turned all the way clockwise until it cannot go anymore to fully shut the tank itself.
  - Remaining air needs to be bled out of regulator.
    - Do this by turning the black regulator dial to allow the air to come out.
    - Watch the dial on the regulator gauge go all the way down to zero.
  - Return the regulator dial back to 0 when the gauge is at zero (red).
- All these need to be done correctly to properly fill a tank.

### **Putting Tanks on Homefill**

- Make sure the green dots are visible on the brass ring on the right.
  - If it isn't, push down on the plastic/metal around it to see it.
- Put brass part of the tank into the brass ring and push down until there is a clicking noise.
  - This ensures the tank is properly connected to the homefill
- Make sure the bottom concentrator is ON and at most 3lpm for a 5LPM concentrator, 8LPM for a 10LPM concentrator.
  - Homefill pulls air from the concentrator to fill tanks through the coil tubing.
    - Make sure the coil tubing is connected to both the concentrator and homefill fully. You will hear a clicking noise when it is fully connected.
- Turn on the homefill using the on/off switch.
  - The homefill will say “below normal” at first. It will switch over to filling when it starts to fill the tank.
    - The amount of time this takes depends on how empty the tank itself is.

### **Taking the Tank off Homefill**

- Turn off the homefill using the on/on switch.
- Push down on the metal/plastic ring on the right until you can see the green dots on brass ring again.
- Lift the tank up and off of the homefill
- Check the fullness by opening the tank and checking gauge.